Transducer Handling Guidelines

70003543 Rev. 0 (ENG)

Ultrasound transducers are sensitive instruments which can easily be damaged by rough handling. Take extreme care when handling or storing transducers. The manufacturer is not liable for damage of the transducer caused by improper use. Any improper use will void the warranty.

Transducer Handling

Take the following precautions with transducers:

- Do not subject the transducer to mechanical shock or impact, which may result in cracks or chips in the housing and degrade performance.
- Do not allow the transducer head to hang freely. Impact to the probe head may result in irreparable damage.
- Do not drop the transducer. Always keep the transducer secure when you do not use it.
- You always use the carrying case when you transport the transducer.
- Using an inappropriate cleaning or disinfecting agent may damage or discolor the transducer. Be sure to use the proper disinfectant and disinfection method recommended by ALPINION MEDICAL SYSTEMS. Refer to Download Center on website (<http://www.alpinion.com/web/support/download.asp>) for the latest list of compatible cleaning solutions and disinfectants.
- The transducer have been designed and tested to be able to withstand high-level disinfection as recommended by the manufacturer of the disinfectant product. Carefully follow the disinfectant manufacturer's instructions.
- To avoid damage to the transducer, do not use transducer sheaths containing an oil-based coating or petroleum- or mineral oil-based ultrasound coupling agents. Use only a water-based ultrasound coupling agent.
- Do not attempt to repair or alter any part of the transducer. Contact your service representative at ALPINION MEDICAL SYSTEMS immediately if a transducer appears to be damaged or malfunctions in any way.

Transducer Cable Handling

Take the following precautions with transducer cables:

- Carefully wind the cable into the carrying case when storing or transporting the transducer.
- Do not attempt to move the console using any cables or fixtures, such as the transducer connectors.
- Do not subject the cables to bending or twisting. Excessive bending or twisting of cables can cause a failure or intermittent operation.
- Do not let transducer cables dangle loosely from the ultrasound system where they might be caught in the casters while moving.
- Do not place the transducer cable on the floor where the cable can be run over by equipment wheels, etc.
- Do leave the cable under any heavy object.
- Avoid crossing cables between transducers.
- Kinks can cause damage to the cable over time. Do not allow the cable to kink.

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